

Sport & Sport Max Front Closure

Application Instructions CP020152 Rev. C 5/00



Council Directive 93/42/EEC
of 14 June 1993 concerning
Medical devices

ENHANCES PROPRIOCEPTION AND JOINT POSITION SENSE FOR MAXIMUM SUPPORT IN A CONVENIENT FRONT CLOSURE FORMAT

From the family whose name means braces



This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient. This device is generally not intended for patients in excess of 250 lbs. (114 kilos).

Indications: Indicated for mild sprains of the medial or lateral collateral ligaments, mild injuries of the menisci, or mild injuries to the patellar retinaculum, to provide increased support, compression and warmth, and aid in proprioception.

Contraindications: Contraindicated for injured or reconstructed knees that have moderate to severe instability.

WARNING: Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. **If any additional pain or symptoms occur while using this device, discontinue use and seek medical attention.**

Warranty: This device is warranted to be free from defects in material and workmanship for a period of 90 days for wraps, pads, straps and normal wear components and 12 months on all other parts. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect.

Manufactured by:

Bledsoe Brace Systems™ by Medical Technology Inc.
2601 Pinewood, Grand Prairie, Texas, 75051, USA
Toll Free Tel. **1-888-BLED SOE** (253-3763)
Or 1-800-527-3666 • Local Tel. 972-647-0884
Local FAX 972-660-5495
International FAX 972-606-0649
www.bledsoebrace.com

European Authorized Representative:
MEDPASS International GmbH
Philippine-Welser Str. 6
86150 Augsburg
Germany

For product information or questions pertaining to sales or service, please contact the national distributor in your area or the manufacturer.

Sport & Sport Max Wrap Front Closure Application Instructions



LOOSEN ALL STRAPS
Step 1. Unfasten all straps and detach from one arm, leaving D-ring end attached to other arm. This allows more freedom when fitting brace to leg.



APPLY NEOPRENE WRAP
Step 2. Without removing hinges or straps, wrap the neoprene sleeve onto the leg until hinges line up with center of knee.



CLOSE LOWER WRAP FASTENER
Step 3. Attach lower hook-to-pile fastener.



CLOSE UPPER WRAP FASTENER
Step 4. Lightly position upper hook-to-pile fastener. If necessary, re-position wrap fasteners for an even fit. **NOTE:** Hinge arms should be checked for alignment prior to securing straps.



SECURE LOWER STRAP
Step 5. Wrap lower strap around brace, then loop strap through D-ring, pull back to tighten, and press to close.



SECURE UPPER STRAP
Step 6. Wrap upper strap around brace, then loop strap through D-ring, pull back to tighten, and press to close. Secure the two remaining straps in the same manner. **NOTE:** The Sport Front Closure has only two straps.