

# Bledsoe Sport & Sport Max



Application Instructions CP020131 Rev. C 5/00

**ENHANCES PROPRIOCEPTION AND JOINT POSITION SENSE FOR MAXIMUM SUPPORT TOTAL LEG SUPPORT SYSTEM WITH INFINITE R.O.M. & SHORT APPLICATION TIME**

*From the family whose name means braces*



This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient. This device is generally not intended for patients in excess of 250 lbs. (114 kilos).

**Indications:** Indicated for mild sprains of the medial or lateral collateral ligaments, mild injuries of the menisci, or mild injuries to the patellar retinaculum, to provide increased support, compression and warmth, and aid in proprioception.

**Contraindications:** Contraindicated for injured or reconstructed knees that have moderate to severe instability.

**WARNING:** Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. **If any additional pain or symptoms occur while using this device, discontinue use and seek medical attention.**

**Warranty:** This device is warranted to be free from defects in material and workmanship for a period of 90 days for wraps, pads, straps and normal wear components and 12 months on all other parts. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect.

Manufactured by:

**Bledsoe Brace Systems**™ by Medical Technology Inc.  
2601 Pinewood, Grand Prairie, Texas, 75051, USA  
Toll Free Tel. **1-888-BLED SOE** (253-3763)  
Or 1-800-527-3666 • Local Tel. 972-647-0884  
Local FAX 972-660-5495  
International FAX 972-606-0649  
[www.bledsoebrace.com](http://www.bledsoebrace.com)

European Authorized Representative:  
**MEDPASS International GmbH**  
Philippine-Welser Str. 6  
86150 Augsburg  
Germany

**For product information or questions pertaining to sales or service, please contact the national distributor in your area or the manufacturer.**

# Bledsoe Sport & Sport Max Application Instructions



## LOOSEN ALL STRAPS

Loosen all straps to their maximum circumference leaving D-ring end attached to hinge arm. This allows more freedom when pulling brace onto leg.

## APPLY NEOPRENE SLEEVE

Without removing hinges or loosened straps, slide the neoprene sleeve onto the leg until hinges line up with center of knee.



## SECURE LOWER STRAP

Wrap lower strap around brace, then loop strap through D-ring. Pull back to tighten, and press to close.

# ***Bledsoe Sport & Sport Max Application Instructions***



## **SECURE ALL STRAPS**

Replace second strap by pressing in on both arms while applying strap tension towards the front of brace. Wrap strap around brace (step 4), then loop strap through D-ring, pull back to tighten (step 4a), and press to close. **NOTE:** Sport Brace has two straps and Sport Max Brace has four straps. (Sport Max is shown)

On Sport Max: repeat steps on straps three and four, alternating front to back.